


ORDER OF THE DRAW

KNOW YOUR FILL LEVEL

- Fill level has +/- 10% tolerance
- Anything above or below the tolerance can impact patient results

Number of Inversions:

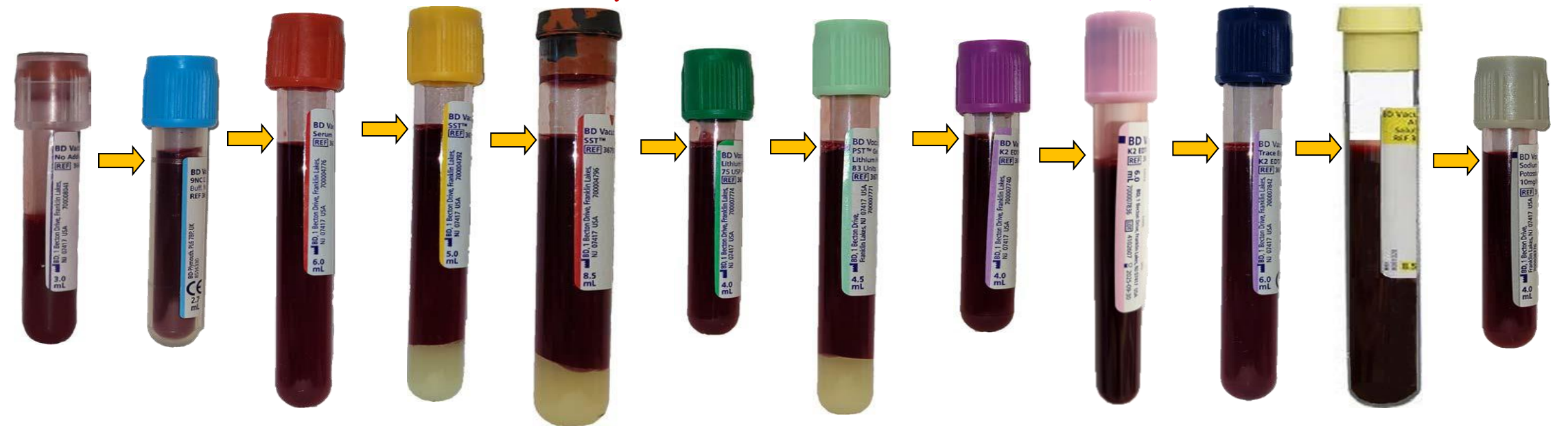

| Colour | Clear with Red inset | Light Blue | Red | Gold | Red/Grey (Tiger Top) | Dark Green | Light Green | Lavender | Pink or Lavender | Royal Blue | Yellow | Grey |
|--------------|-----------------------------------|--------------|-----------------------------|--|-----------------------|---|-------------------------------------|---|---------------------------------|--|--|-------------------------|
| Additive | No additive | Na-Citrate | Clot activator | Clot activator + Gel | Clot activator + Gel | Li-Heparin, or Na-Heparin (striped label) | Li-Heparin + Gel | K ₂ EDTA 7.2 mg | K ₂ EDTA 10.8 mg | K ₂ EDTA 10.8 mg Metal-Free | Acid Citrate Dextrose (ACD) | Na-Fluoride + K-Oxalate |
| Common Tests | None (discard tube), iSTAT INR | PT/INR, aPTT | Immunology tests, Vitamin D | Immunology tests, Bile Acids, Lithium, Ionized Calcium | Serology (Cadham Lab) | <u>Li-Heparin:</u> Drug levels, Ethanol, iSTAT CHEM8 & Blood Gas <u>Na-Heparin:</u> Cytogenetics (karyotype) | <i>Most</i> general Chemistry tests | CBC, Flow cytometry, FK5, Sirolimus, Tacrolimus, ACTH, PCR Viral Loads (Cadham Lab) | Type & Screen, Crossmatch (CBS) | Trace metals | HLA Crossmatch (transplants), DNA analyses | Lactate |

FULL TUBES = QUALITY RESULTS