



Shared health
Soins communs
Manitoba

Daily Cleaning Log Sheet: Abbott ID NOW™ Instrument and Work Area

Document # F120-10-59A

Version # 03

Approved By:
J. Carlson
approval on file

Effective Date

13-JAN-2022

Source Document:

Abbott ID NOW COVID-19 Procedure Manual,
doc# 120-10-59

Daily Cleaning Log Sheet: Abbott ID NOW™ Instrument and Work Area

Site: _____ Serial #: _____ Year: _____

Instructions for Daily Cleaning:

- Wear appropriate PPE
- Power off the Abbott ID NOW™ Instrument and the printer
- Wipe all outer surfaces of the instrument with appropriate material (Acceptable cleaning agents include: 70% ethanol, 70% isopropanol, 10%bleach (prepare daily))
- Open the instrument lid and wipe down the visible surfaces
- Change the absorbent paper/pad on the workspace
- Wipe down the bench area
- Discard used paper products into biohazard waste

Initial each day when cleaning is complete

Month	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31
January																															
February																															
March																															
April																															
May																															
June																															
July																															
August																															
September																															
October																															
November																															
December																															

Reviewed by (CT or delegate): _____

Date: _____