


	 <p>Shared health Soins communs Manitoba</p> <p>Standard Operating Procedure (SOP) for: Disinfecting High Impact Safety Glasses (Re-usable) for Security Services</p>	<p>Status: July 29, 2022</p> <p>Revised: Owner: Shared Health</p>
<p>Disinfecting High Impact Safety Glasses (Re-usable) for Security Services</p>		<p>Page 1 of 4</p>

Purpose	This SOP pertains to disinfecting high impact safety glasses for Security Services.	
Background	<p>Product name: Uvex High Impact Protector Product code: S3960HS</p>	
Required Materials	<p>Facility-approved disinfectant wipes Disposable gloves</p>	
Beginning of shift	<p>Take new/disinfected safety glasses for use during shift. Note: dedicated, single person use, product</p>	

The Process:

Step	Instructions: When removed during the shift
1	Perform hand hygiene.
2	Remove eye protection. Secure glasses by grasping one side of frame. Use other hand to grasp strap. Gently pull strap away from head and lift off eye protection.
3	Grasp one side of glasses frame. Clean and disinfect all surfaces including strap with facility approved disinfectant .
4	<p>Complete cleaning and disinfection holding the safety glasses in the air by the disinfectant wipe and allow the recommended contact time. The other option is to place the safety glasses on a clean surface, on top of a new disinfectant wipe, and allow the recommended contact time.</p> <p>Note: Intervention Disinfectant = 1-minute wet contact time.</p>
5	Perform hand hygiene.
6	After the wet contact time achieved allow safety glasses to fully dry in a clean location.
7	Store clean/disinfected safety glasses in a manner to prevent contamination once dry (e.g. Ziploc bag).

Note: *Inspect for clarity after cleaning/disinfection completed. Rinse with potable water if concerns with clarity/clouding, and dry completely prior to storage. Discard and replace when cracking, pitting, or scratches reduce vision.*

Protocol Algorithm for Disinfecting High Impact Safety Glasses

Beginning of shift:

Take a new/disinfected Safety Glasses for use during shift

When removed during the shift follow the process below.

Step 1:

Perform hand hygiene.

Step 2:

Remove eye protection. Secure glasses by grasping one side of frame. Use other hand to grasp strap. Gently pull strap away from head and lift off eye protection.

Step 3:

Grasp one side of glasses frame. Clean and disinfect all surfaces including strap with facility approved disinfectant.

Step 4:

Complete cleaning and disinfection holding the safety glasses in the air by the disinfectant wipe and allow the recommended contact time. The other option is to place the safety glasses on a clean surface, on top of a new disinfectant wipe, and allow the recommended contact time.

Step 5:

Perform hand hygiene.

Step 6:

After the wet contact time achieved allow safety glasses to fully dry in a clean location.

Step 7:

Store clean/disinfected safety glasses in a manner to prevent contamination once dry (e.g. Ziploc bag).



Shared health
Soins communs
Manitoba

**Standard Operating
Procedure (SOP) for:
Disinfecting High Impact
Safety Glasses (Re-usable)
for Security Services**

Status: July 29, 2022

Revised:
Owner: Shared Health

Disinfecting High Impact Safety Glasses (Re-usable) for Security Services

Page 3 of 4

Step	When Visibly Soiled
1	Perform hand hygiene and don gloves.
2	Remove as outlined in Step # 2 above.
3	Disassemble glasses by removing lens using dark gray tabs on either side of frame.
4	Gently lift tab on one side and slide out lens. Repeat on other side.
5	Thoroughly clean and disinfect all parts of the eye protection including lens, frame and strap with disinfectant wipe.
6	If unable to completely remove soil, discard and obtain new pair.
7	Remove gloves and perform hand hygiene.
8	Ensure all pieces of eye protection are completely dry before putting back together.
9	Store clean/disinfected safety glasses in a manner to prevent contamination once dry (e.g. Ziploc bag).

Protocol Algorithm for Disinfecting High Impact Safety Glasses

**Beginning of shift:
Take a new/disinfected Safety Glasses for use during shift**

When visibly soiled follow the process below.

