

Document #:	 Standard Operating Procedure (SOP)	Status: v 1.0 Revised: September 20, 2021 Owner: Program Director, Infection Prevention and Control
CADD®-Solis Epidural Infusion Pump Cleaning & Disinfecting SOP		Page 1 of 2


Purpose	To outline requirements for cleaning and disinfection of infusion pumps. Infusion pumps must be cleaned and disinfected after each patient use and as necessary (e.g., if visibly soiled).
Background	CADD Solis infusion pump must be cleaned and disinfected after each patient use and as necessary (e.g., if visibly soiled).
Required Materials	Facility-approved disinfectant wipes. Where wipes are not available, ready-to-use (RTU) disinfectant solution with soft, lint-free cleaning cloths. NOTE – Accel Intervention wipes and/or Accel Intervention RTU solution are <u>NOT</u> to be used on CADD®-Solis Epidural Infusion Pumps.



The Process:

Step	1) Inspect the Downstream Occlusion (DSO) seal for damage or wear to the rubber material during cleaning and at each preventative maintenance service.
1.1	DSO seals that show signs of cracking or abrasion(s) should be returned to a Smiths Medical Service Center for repair. If the DSO seal does not have cracking and/or abrasion(s), continue with the following steps to clean and disinfect the pump.

Document #:	 Standard Operating Procedure (SOP)	Status: v 1.0 Revised: September 20, 2021 Owner: Program Director, Infection Prevention and Control
CADD®-Solis Epidural Infusion Pump Cleaning & Disinfecting SOP		Page 2 of 2

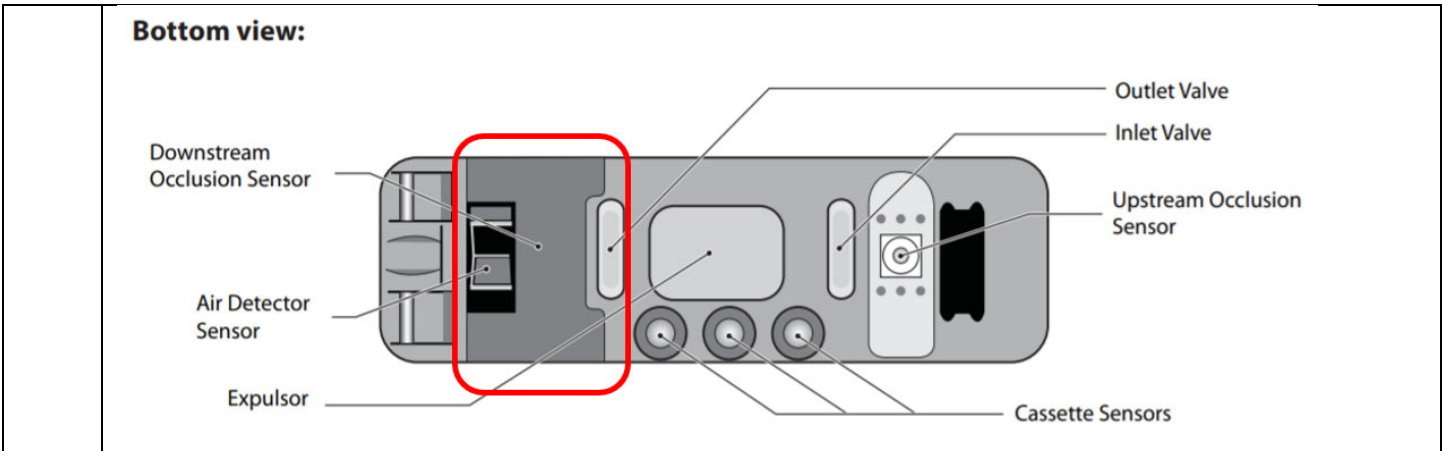

Step	2) Clean and disinfect the pump and lockbox with facility-approved disinfectant wipes.
2.1	Before cleaning and disinfecting the pump, always disconnect the pump from the patient, switch off the device, and unplug from power outlet.
2.2	Remove any visible residue or contaminated material from all surfaces.
2.3	Wipe all surfaces of the pump and lockbox with facility-approved disinfectant.
2.4	Follow the disinfectant manufacturer's recommendations for disinfectant contact times.
2.5	Do not spray disinfectants directly on the pump. Do not allow the solution to soak into the pump.
Step	3) Allow the pump and lockbox to dry completely before use.

Acceptable disinfecting solutions for the CADD®-Solis Epidural Infusion Pump are listed below:	
<ul style="list-style-type: none"> • CaviWipes 	