

	 <p>Standard Operating Procedure (SOP) for: Disinfecting Safety Glasses (Re-usable)</p>	<p>Status: v 1 Revised: June 24 2022 Owner: Shared Health</p>
Disinfecting Safety Glasses (Re-usable)		Page 1 of 2

The Process:

	<p>Instructions This SOP pertains to disinfecting safety glasses:</p> <table border="0" data-bbox="212 609 1570 856"> <tr> <td data-bbox="212 609 487 777">Product name</td> <td data-bbox="487 609 1006 777">MCR Safety Crews Tomahawk</td> <td data-bbox="1006 609 1570 777">Pyramex OTS Anti-Fog Safety Glasses</td> </tr> <tr> <td data-bbox="212 777 487 856">Product code</td> <td data-bbox="487 777 1006 856">TK110</td> <td data-bbox="1006 777 1570 856">S3510STJ</td> </tr> </table>		Product name	MCR Safety Crews Tomahawk	Pyramex OTS Anti-Fog Safety Glasses	Product code	TK110	S3510STJ
Product name	MCR Safety Crews Tomahawk	Pyramex OTS Anti-Fog Safety Glasses						
Product code	TK110	S3510STJ						
	<p>Beginning of shift: Take new/disinfected safety glasses for use during shift. Note: dedicated, single person use, product</p>							

Step	Instructions: When removed during the shift
1	Perform hand hygiene.
2	Remove eye protection. Pull the arms out to the 'fully extended' position. <i>If grossly/visibly soiled perform hand hygiene and don gloves first, remove, then dispose of eye wear and take new safety glasses.</i> ,
3	Grasp one arm of the safety glasses frame. Clean and disinfect the complete surface and arms of the safety glasses with a facility approved disinfectant.,
4	Complete cleaning and disinfection holding the safety glasses in the air by the disinfectant wipe and allow the recommended contact time. The other option is to place the safety glasses on a clean surface, on top of a new disinfectant wipe, and allow the recommended contact time. Note: Intervention Disinfectant = 1-minute wet contact time
6	Perform hand hygiene
7	After the wet contact time achieved allow safety glasses to fully dry in a clean location
8	Store clean/disinfected safety glasses in a manner to prevent contamination once dry (e.g., Ziploc bag).

Note: inspect for clarity after cleaning/disinfection completed. Rinse with potable water if concerns with clarity/clouding, and dry completely prior to storage. Discard and replace when cracking, pitting, or scratches reduce vision

Protocol Algorithm for Disinfecting Safety Glasses

Beginning of shift:
Take a new/disinfected Safety Glasses for use during shift

When removed during the shift follow the process below

Step 1:
Perform hand hygiene

Step 2:
Remove eye protection. *If grossly/visibly soiled, don gloves first and then dispose. Take new safety glasses*

Step 3:
Pull the arms out to the 'Fully extended' position. Grasp one arm of the safety glasses frame

Step 4:
Clean and disinfect the complete surface and arms of the safety glasses with a facility approved disinfectant

Step 5:
Complete cleaning and disinfection holding safety glasses in the air by the disinfectant wipe and allow recommended wet contact time

Step 5:
Place safety glasses on a clean surface, on top of a new disinfectant wipe. Allow recommended wet contact time

Step 6:
Perform hand hygiene

Step 7:
After wet contact time achieved allow safety glasses to fully dry in a clean location

Step 8:
Store clean frame in a manner to prevent contamination once dry (e.g., Ziploc bag)