



<b>Document #:</b>  <b>CO-1</b>	 <b>Standard Operating Procedure (SOP) for: Disinfecting Eye Protection (re-usable frame and disposable lens)</b>	Status: v 1.3  Revised: June 21, 2022 Owner: Shared Health
<b>Disinfecting Eye Protection (re-usable frames and disposable lens)</b>		Page 1 of 2

## The Process:

	<b>Instructions</b> This SOP pertains to Disinfecting Eye Protection (re-usable frame and disposable lens). <ul style="list-style-type: none"> <li>Adhere to recommended manufacturer instructions for cleaning and disinfection.</li> <li>Consider the following when manufacturer instructions for cleaning and disinfection are unavailable (such as for single use disposable eye protection).</li> </ul>
	<b>Beginning of shift:</b> Take a new/disinfected/reprocessed frame and new lens for the eye protection.

Step	Instructions: When removed during the shift
1	Perform hand hygiene.
2	Remove eye protection. <i>If grossly/visibly soiled, perform hand hygiene and don gloves first.</i>
3	Carefully remove the lens from the frame.
4	Dispose of the lens.
5	Clean and disinfect the complete surface of the frame using a facility approved disinfectant wipe (e.g., Accel Intervention).
6	Grasp one end of the frame with the disinfectant wipe. Hold in air to achieve required wet contact time (Accel Intervention is 1 minute). Dispose of the frame if visibly soiled.
7	Place on paper towel or clean surface. Allow to air dry before reusing.
8	Perform hand hygiene.
9	Store cleaned/disinfected eye frame in a manner to prevent contamination once dry (e.g., Ziploc bag).

<b>Document #:</b>  <b>CO-1</b>	 <b>Standard Operating Procedure (SOP) for: Disinfecting Eye Protection (re-usable frame and disposable lens)</b>	<b>Status:</b> v 1.3 <b>Revised:</b> June 21, 2022 <b>Owner:</b> Shared Health
<b>Disinfecting Eye Protection (re-usable frames and disposable lens)</b>		<b>Page 2 of 2</b>

**Protocol Algorithm for Disinfecting Eye Protection (re-usable frame and disposable lens)**

**Beginning of shift:**  
**Take a new frame and new lens for the eye protection.**

**When removed during the shift follow the process below**

