SWP-MMH-CLEANING OF SLEIGH BEDS



Description of Task: Cleaning of Sleigh Beds							
Position/Job: 1 Housekeeping personnel			Department/Unit: Adult mental Health units				
Potential Hazards			Personal Protective Equipment Required				
	Risk for Injury (check all that apply)		Spec		e, Manufacturer, Model		
	Biological (puncture/spill/splash/exposu	re)		Head	Protection:		
\boxtimes	Caught In/Under/Between Wall, Equipm	nent		Eye P	rotection:		
\boxtimes	Chemical/Hazardous Substance/Medica	ation		Hearir	ng Protection:		
	Contact with Hot/Cold Object/Substance	е		Face I	Protection:		
	Contact with Sharp Object (not biological	al)	\boxtimes	Hand	Protection:		
	Electrical		\boxtimes	Foot F	Protection:		
	Falling or moving equipment/material			Respi	rator/Fit Test:		
	Moving machinery / parts			Body	Protection:		
\boxtimes	MSI-Awkward / sustained postures			Hi Vis	ibility Apparel:		
\boxtimes	MSI-Forceful exertions		\boxtimes	Other:	isolation precautions it	f required	
\boxtimes	MSI-Repetitive motions		Equipment and/or tools required to perform task				
	□ Noise Cleaning supplies						
\boxtimes	Surfaces/Objects causing slips, trip or falls			Knee pads and/or long handled reacher			
	Task requires more than 1 worker						
	Violent, Aggressive or Reactive Behaviours For machine operation tasks complete the following				plete the		
				Machine Description and Safety Features:			
	1 1 1 1 1 1 1 1 1 1 1 1 1 1 1 1 1 1 1 1						
\boxtimes	Other: Pinch Points						
Patient/Client/Resident Related Tasks - be			Set-up and maintenance of this equipment is only to				
knowledgeable of patient specific hazards (CARE			be performed by trained/authorized staff following				
, , , , , , , , , , , , , , , , , , , ,				the manufacturer's manual.			
Isolation Precautions, etc.)							
Training and Proficiency: Staff who perform this task must be trained as indicated below in this safe work procedure prior to							
performing it. Training must be documented. This SWP must be available to staff.							
⊠Read Procedure and Sign □Other Required Training: <i>Specify</i>							
□Demonstrated Competency							
Responsibilities							
Man	ager/Supervisor or Designate: Ensure	all staff			ming task: Perform tas		
are trained and that duties are performed in			to training, established health and safety regulations,				
accordance to training, established health and safety			guidelines, policies and procedures (e.g. following				
regulations, guidelines, policies and procedures (e.g. following safe work procedures).			safe work procedures). Follow process for reporting hazards, injuries, occurrences and patient safety				
Tollowing sale work procedures).			events.				
Approved by Original					Revised Effective	Reviewed Date	
Date				Date			
Provincial Workplace Safety & Health Februa			ary 3, <i>1</i>	2023	February 3, 2023	February 3,	
Wor	king Group				2023		

SWP-MMH-CLEANING OF SLEIGH BEDS



This safe work procedure was created by Provincial Safe Work Procedure Committee and will be reviewed any time the task, equipment or materials change and at a minimum of every three years from the last revision date.

, ,	ne the task, equipment or materials change and at a minimum of every three years from the last on date.				
	/Other Considerations:				
Steps to be taken to complete task safely:					
Remove mattress from bed					
1	Remove linens if not already done. Do this by moving around bed to reduce reach and bending knees to get lower as needed.				
2	Using a wide base of support, back straight, and elbows close by side. Slowly push the mattress towards the wall by shifting your weight from back foot to front foot. Walk to other side to move mattress off the bed to lean against the wall.				
	Clean bed				
1	Wear gloves and use hospital approved disinfectant to clean and disinfect all exposed surfaces. Knee pads may be used to allow worker to go down on one knee to lower down to wash lower bed parts. A long handled reacher can also be used to reduce reaching to the surface if not kneeling. Move around the bed and bend knees to reduce bending and reaching.				
	Clean Mattress				
1	Move around mattress to clean and disinfect top 2/3 of mattress while mattress is standing against wall.				
2	Bend through hips and knees and weight shift to place mattress on the bed with bottom third hanging off the bed frame. Clean the remaining mattress and then move it onto the clean bed.				
	Clean under bed as part of regular floor washing				
1	With arm fairly straight, bend knees in lunge position or use one knee down with knee pad to lower broom handle to get under bed. *insert picture*				
	Related Materials:				
Refer	References:				