


Shared Health – Emergency Response Service

General - Safe Work Procedure



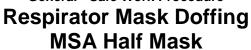

Hazards Present:	Personal Protective Equipment:		Minimum Training Requirement:	
Biohazard	None		N/A	
Purpose:		Indications:		
To provide personal protection from potentially contaminated surfaces of respirator		Respirator no longer required		
Equipment:		Contraindications:		
MSA Half Mask Respirator		None		

Safe Work Procedure

- 1. Observe Routine Practices and use proper body mechanics
- 2. Perform hand hygiene using an alcohol based hand sanitizer
- 3. Loosen the neck strap and undo the clasp
- 4. With head tilted forward, slide the cradle headband forward over your head while maintaining control of respirator
- 5. Remove respirator from face
- 6. Perform hand hygiene using an alcohol based hand sanitizer
- 7. Clean the face mask, following the Safe Work Procedure Cleaning MSA Half Mask Respirator

Steps for Doffing MSA Advantage Respirator Half Mask	Important Points
Observe Routine Practices and use proper body mechanics	Following Routine Practices can reduce the transmission of pathogens and other microorganisms
2. Perform Hand Hygiene	Use and alcohol based hand sanitizer



Shared Health – Emergency Response Service

General - Safe Work Procedure

Respirator Mask Doffing MSA Half Mask



3.	Loosen the neck strap and undo the clasp.		
4.	With head tilted slightly forward, slide the cradle headband forward over your head while maintaining control of respirator.		
5.	Remove respirator from face.		
6.	Perform Hand Hygiene	•	Use an alcohol based hand sanitizer



Shared Health – Emergency Response Service

General - Safe Work Procedure





Created with assistance from WFPS
Training Reviewed: April 3, 2020
IP&C reviewed: 04/28/2020
OESH reviewed: 04/28/2020
Approved by : 04/28/2020
Implemented: 04/29/2020
Revised: 02/10/2021