









**Appendix 12 Order of Draw Poster (Continued)**

Micro-Collections via Skin Puncture

**Order of Draw**

Closure Color	Additive	Mix by Inverting
 Lavender	K <sub>2</sub> EDTA	10x
 Green	Lithium Heparin	10x
 Mint Green	Lithium Heparin and Gel for plasma separation	10x
 Mint Green		
 Grey	NaF/ Na <sub>2</sub> EDTA	10x
 Gold	Clot Activator and Gel for serum separation	5x
 Gold		
 Red	No additive	0x